

Nursing aids

The design of our models is protected against counterfeiting. TS

Positioning cushions

Positioning cushions are used to support parts of the body or the whole body and to place the body in the desired position.

Quality 1010 : Breathable, moisture-repellent and flame-retardant fabric with high tear resistance.



Md 40641 TS

Cover for the boomerang XS pillow, outer circumference +/- 120 cm, in quality 1010 with zipper.

 Washing instructions for quality 1010:

 1. Washing
 Wash separately at 60°C with little detergent.

 2. Rinse
 DO NOT ADD FABRIC CONDITIONER.

 3. Dry
 Dry very briefly at low temperature in the dryer or hang to dry. Do not dry together with moisture-retaining textiles (like cotton,...).

anne-]ore

Van Herck bvba | Tel 0032(0) 14-58.84.80 | Fax 0032(0) 14-58.09.50 | www.van-herck.be | mail: anne-lore@van-herck.be